

## Aptima Assay Fluids Kit (Panther™ System)

For *in vitro* diagnostic use.

### Intended Use

The Aptima Assay Fluids Kit is for use on the Panther System.

### Materials Provided

**Note:** For information on any hazard and precautionary statements that may be associated with reagents, refer to the Safety Data Sheet Library at [www.hologic.com/sds](http://www.hologic.com/sds).

#### Aptima Assay Fluids Kit (Cat No. 303014)

(Store at 15°C to 30°C upon receipt)

Symbol	Component	Quantity
W	Aptima Wash Solution <i>10 mM HEPES buffered solution containing less than 2% preservative.</i>	1 x 2.9 L
O	Aptima Oil Reagent <i>Silicone oil.</i>	1 x 260 mL
DF	Aptima Buffer for Deactivation Fluid <i>800 mM Bicarbonate buffered solution.</i>	1 x 1.4 L

### Warnings and Precautions

- A. Use routine laboratory precautions.
- B. Avoid microbial and ribonuclease contamination of Aptima Assay Fluids.

### Storage and Handling Requirements

- A. Store unopened Aptima Assay Fluids at 15°C to 30°C.
- B. Once opened, Aptima Assay Fluids are stable for 60 days.
- C. Do not use after the expiration date.

### Procedure

Replace bottles in the Universal Fluids Drawer when notified by the system. Refer to the *Panther System Operator's Manual*.



Hologic, Inc.  
10210 Genetic Center Drive  
San Diego, CA 92121 USA

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

Customer Support: +1 844 Hologic (+1 844 465 6442)  
customersupport@hologic.com

Technical Support: +1 888 484 4747  
molecularsupport@hologic.com

For more contact information visit [www.hologic.com](http://www.hologic.com).

Hologic, Aptima, and Panther are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All other trademarks that may appear in this package insert are the property of their respective owners.

©2001–2014 Hologic, Inc. All rights reserved.

502181EN Rev. 001  
2014-11